UpDate!

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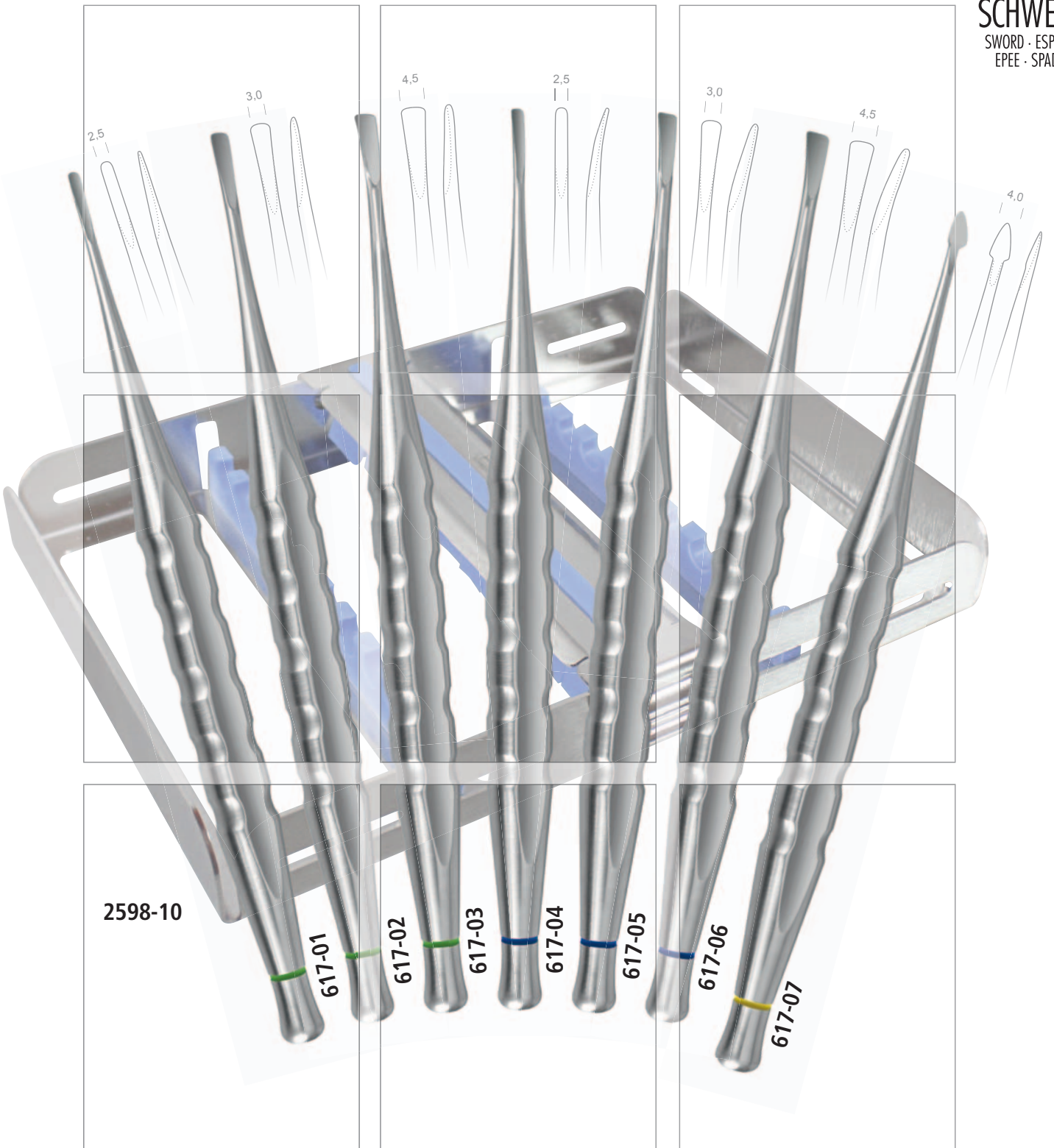
P-LUX

MODERN THERAPY - MODERN INSTRUMENTS

SCHWERT
SWORD · ESPADA
EPEE · SPADA

SCHWERT
SWORD · ESPADA
EPEE · SPADA

P-Lux

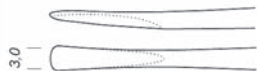
Modern Therapy · Modern Instruments

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P-LUX Fig. 1, 2,5 mm, straight, green,
for front teeth and upper jaw

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P-LUX Fig. 2, 3,0 mm, straight, green,
for front teeth and upper jaw

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P-LUX Fig. 3, 4,5 mm, straight, green,
for front teeth and upper jaw

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P-LUX Fig. 5, 3,0 mm, curved, blue,
for lower jaw

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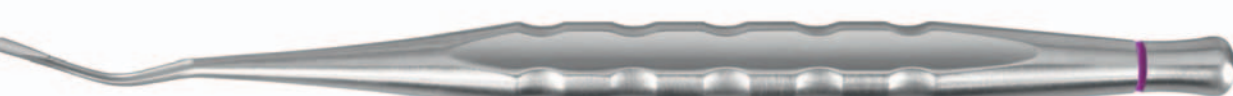
P-LUX Fig. 6, 4,5 mm, curved, blue,
for lower jaw

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P-LUX Fig. 7, desmotime, straight, for
initial loosening of the desmodontal fibres

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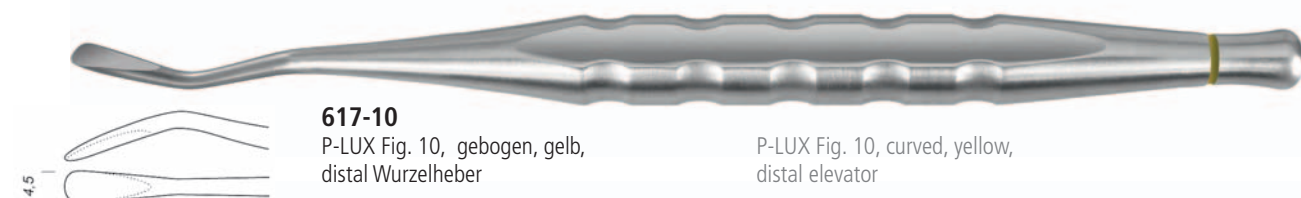
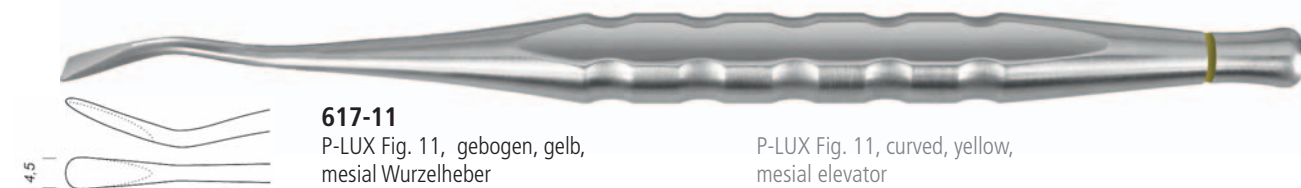
P-LUX Fig. 10, curved, yellow,
distal elevator

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P-LUX Fig. 11, curved, yellow,
mesial elevator

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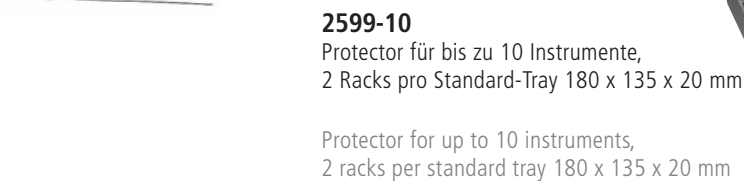
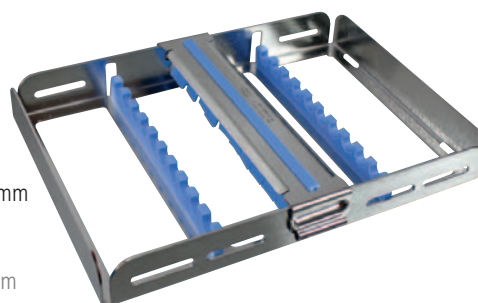
Protector for up to 10 instruments,
2 racks per standard tray 180 x 135 x 20 mm

SCHWERT
SWORD · ESPADA
EPEE · SPADA

P-Lux

Modern Therapy · Modern Instruments

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The new P-LUX Extraction Instruments are the modern, universal and complete extraction instrument series in one tray.

The new handle design is highly ergonomic and prevents unintended slipping during the usage. It also guarantees a direct and controlled power transmission to prevent tooth and root fractures. All instruments are perfectly suited for a non-traumatic tooth extraction without injuring surrounding structures. The colour coding allows a clear handling.

1. Initial loosening of the desmodontal fibres. Apply the P-LUX Desmotome first approximal then buccal and palatinal. Working position (pencil handle).

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2. Then progressive luxation with the P-LUX instruments (first narrow then wide). Ideal positions: Approximal (mesial or distal) to the tooth which should be extracted!

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3. Complete luxation with support of a wide P-LUX instrument. Gentle extraction without root fractures!

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4. Inspection of the empty tooth socket. Important: Complete removal of all inflammable and connective tissue structures.

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