


INSTRUCTIONS FOR USE

STAINBUSTER® latch-head burs are used at chairside with a **contra-angle** and a water spray which all practitioners have in their dental office and require no special device.

STAINBUSTER® burs should be used at **slow speed**, around 10,000 rpm, applying slight pressure and with a fine water spray.

The water spray carries away the grinding debris, cleans up the working field and enhances visibility and control. Note that using **STAINBUSTER®** burs without water spray is in no way harmful or contraindicated at low-speed: in this case of course, at the end of the polishing session, spraying and rinsing out of the instrumented area is a routine procedure to remove any remaining debris.

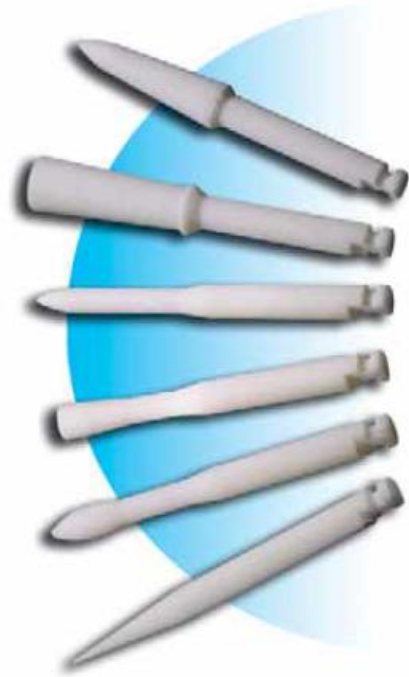

STAINBUSTER® burs can be used as **disposable burs**, one bur per patient, depending on the work being done and the shape used.

The pointed shape, essential for reaching difficult and very narrow places, wears more quickly. It can subsequently be used in more accessible places, **for its abrasive power remains intact even if the initial shape is lost.**

